

# Famos Class 6 Indicator

## Analysis Sheet

## Product Information



### Standard Unprocessed Indicator

- Note 1. Indicator is designed only for use with steam
- Note 2. Green unprocessed indicator and grey processed reference colour
- Note 3. Indicator can be used at 121°C & 134°C
- Note 4. Expiry date for use



### Standard Processed Indicator

- Note 1. Indicator completely changed from green to grey (density of grey may vary)
- Note 2. Indicator and reference not perfectly the same colour
- Note 3. No evidence of lifting laminate




### Indicator Failure

- Note 1. No strong colour change from green to grey

### Other Examples Indicator Pass

- Note 1. Laminate has lifted on edges of indicator
- Note 2. No evidence of green on indicator
- Note 3. Blotchy colour changed caused by steam penetrating laminate and surface of indicator, suggesting damp steam at some stage of the cycle

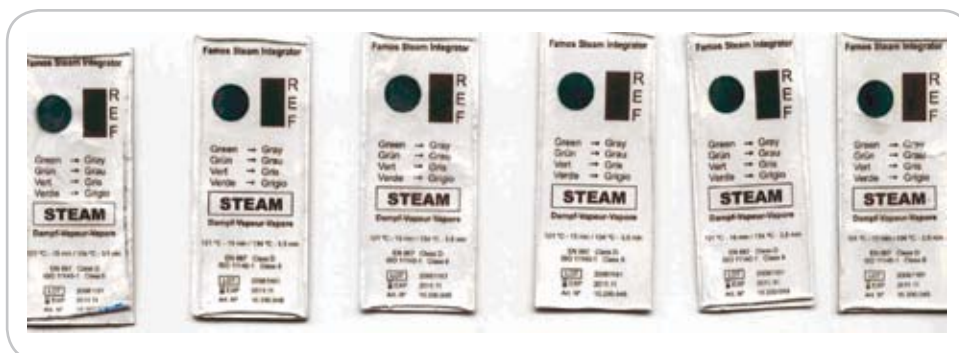

All evaluations are made to conform with EN867 and ISO 11140 Part 1 Class 6

E&OE First Published: 18 December 2008  
Last Updated: 5 November 2010  
File Name: flyer\_CLASS6-INDICATOR-ANALYSIS-SHEET

