



CLEANING INSTRUCTIONS

FOR GREEM SYSTEM REUSABLE LARYNGOSCOPE HANDLE

Laryngoscope Battery Handle:

Battery handles consists of two sections; the main handle and the lamp holder cartridge assembly. The lamp holder cartridge assembly and batteries must be removed prior to disinfection/sterilization.

Procedure:

Main Handle Section:

Remove the batteries and lamp cartridge. The main handle section may then be cleaned with a mild detergent and warm water solution.

Lamp Cartridge:

The lamp cartridge may be wiped clean with a mild detergent and water solution. Do not allow solution to enter into cartridge.

Disinfecting / Sterilization:

Main Handle Section:

Remove batteries and lamp cartridge before subjecting handle to any of the following process. Steam autoclave is appropriate. Do not exceed temperature of 138 °C and pressure of 28 p.s.i. Do not flash autoclave. Ethylene Oxide is appropriate with gas concentrations of 10% - 100% do not exceed 55 °C. Maximum 4 hours.

Exposure time. Aerate 12 – 16 hours at 49 °C following processes.

Lamp Cartridge:

Lamp cartridge can be wiped with cloth dampened with 70% isopropyl alcohol. The lamp cartridge cannot be soaked or autoclaved.