


CALZURO COMFORT HEEL-STRAP: INSTRUCTION SHEET


EUROPEAN PATENTED SYSTEM PD98U000068

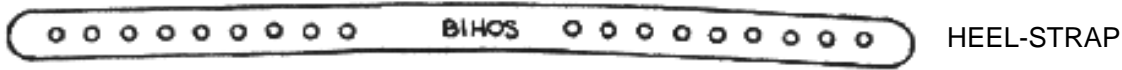

LITTLE MUSHROOMS




FIXING WASHER

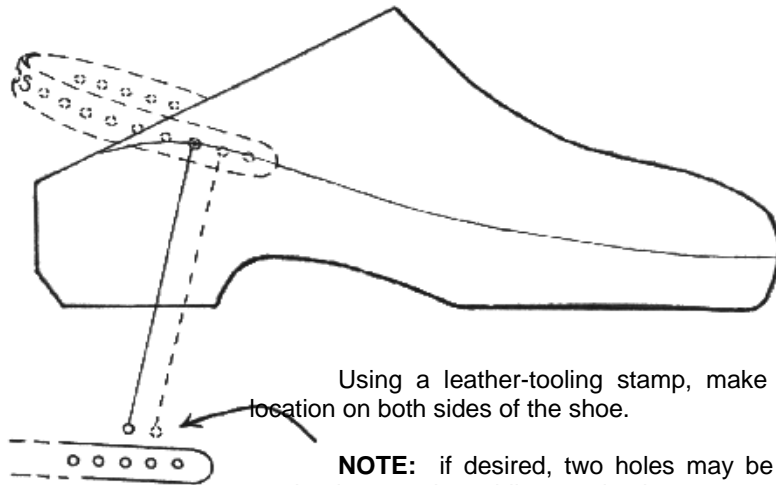

Using a leather-tooling stamp, make a hole 3,2 mm in diameter in the same location on both sides of the shoe.

NOTE: if desired, two holes may be made on one side of Calzuro in order to render the strap immobile once in place.

- Insert the "little mushrooms", head first, from the inside of the shoe.
- Press strap firmly onto the outside head of the "little mushrooms", keeping your heel size in mind (Note: writing should remain on the outside of the strap).
- Press fixing washer firmly between the head of "little mushrooms" and heel-strap.

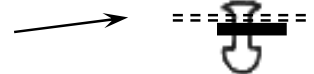

Note: Both heel-strap and "little mushrooms" are autoclavable till 134°C (138°C real)


Standard Kit: 2 heel-straps + 6 "little mushrooms"+ 6 fixing washers.

With reference to the heel-strap to apply on the Calzuro, we state the following:

1. This is an accessory of a CE certified device, manufactured using the same material, so that this follows Calzuro and is according to CE certification.
2. This product is according to Rule 626, specifically to EN-347 Directives.

Suggestion for correctly punching holes in the Calzuro:

We recommend lining up the strap in the desired position to use existing holes in strap as guides through which to punch holes in the Calzuro.

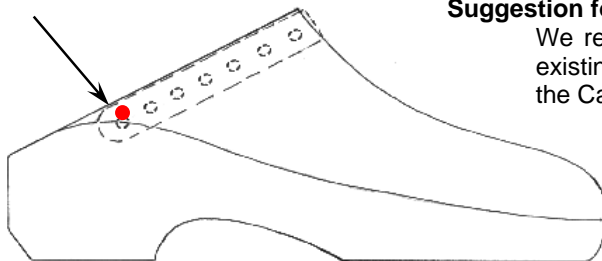

N.B. Once Hole has been punched for heel strap, the shoes are no longer returnable.

Place the heel-strap as shown in the picture and punch the first two holes on both sides of the shoe in the last holes of the heel-strap itself, crossing the slight line onto the side of the shoe.

There is a kit in each single plastic bag.