

F103 TI Heat Sealer

Instructions For Use

- Turn on unit and set the desired temperature using the black + and – buttons.
- Adjust the left hand feed guide to the desired depth.
- Once the temperature as displayed in the red LED readout has reached the set temperature, sealing can be performed. (On initial start-up, the displayed temperature may overshoot the set temperature by up to 6 degrees. Please wait until the temperature has stabilised to within +/- 3 degrees of the set temperature before commencement of sealing)

Note: continue to check the display to ensure the temperature is within +/- 3 degrees of the set temperature before inserting the pack to seal.

- Turn on the top transport motor switch to engage the transport mechanism.
- Insert pack with the end to be sealed into the left hand feed and support the pack as it passes through the sealer.
- If necessary, the reverse transport (lower button) can be pressed to reverse the transport if you wish to abort the seal.
- Turn off transport motor switch between batches of sealing, when not in use or while the sealer is unattended.