

# Correct Use of Stethoscopes

*Instructions for using the Liberty Stethoscopes  
Head Frame and Chest Piece are included in this leaflet.*




The Liberty® Health seal of quality ensures that the product you purchase is free from manufacturing or material faults and has a performance that you would expect for use by a health care professional

## Instructions for Correct Use


### Head Frame

Before placing the ear tips of the head frame in your ears, ensure the ear tips face away from you as illustrated (see diagram A below).

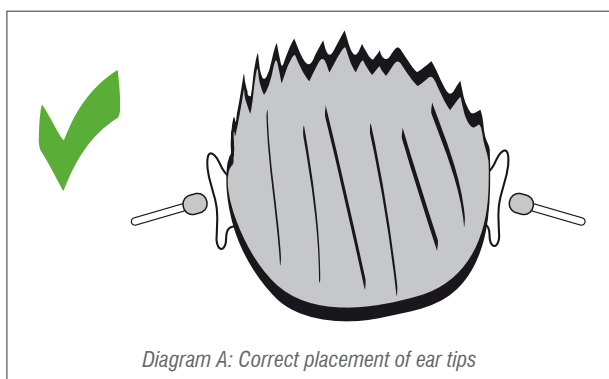

Once seated, you may need to adjust each tip to ensure a proper fit and seal to the ear canal.


Placing the tips in your ears when they face towards you will result in minimal or no sound as this is not anatomically aligned to the ear canal (see diagram B below).

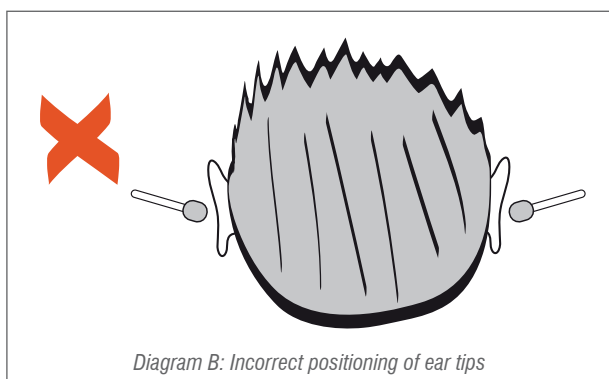

### Dual Head Chest Piece

Dual head stethoscopes have two (2) sides: the diaphragm side and the open conical bell side.

**Important to Note:**  
Only one side is active at any one time.

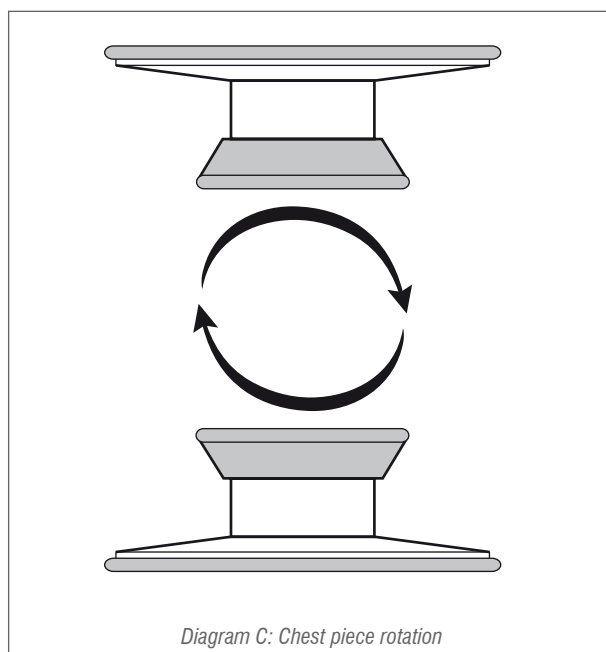

Rotate the chest piece until a click is felt. Test to see which side is active (sounds can be clearly heard from the active side) and if this is not the correct side you wish to use, grasp the stem of the chest piece to which the tubing is attached and rotate 180 degrees until it clicks into position.

**Important to Note:**  
Any position, other than a clicked position, will result in minimal or no sound.

**Disclaimer:** This information sheet is not intended to replace professional instructions and training and should be used as a guide only.