


Rovers® Cervex-Brush® Combi


Instructions for use

Contra-indications: The Rovers® Cervex-Brush® Combi should not be used during pregnancy.


1. Please follow these directions to be assured of the best performance from the Rovers® Cervex-Brush® Combi.

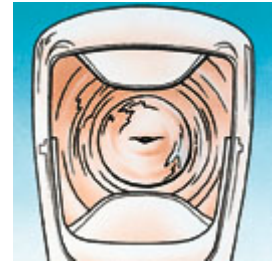

2. Insert the central bristles of the brush into the endocervical canal. Use gentle pressure on the cervix until the lateral bristles bend against the ectocervix

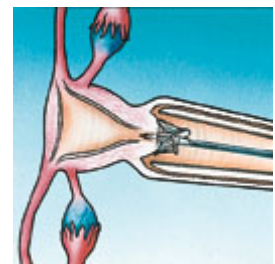

3. Maintain the gentle pressure and rotate the Rovers® Cervex-Brush® Combi **two** times in a clockwise direction by rolling the stem between thumb and forefinger.

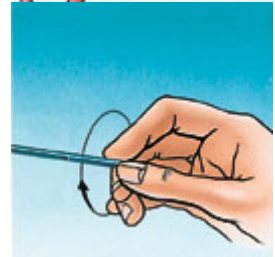

Choose the preferred method. (A)

4a. Insert the Rovers® Cervex-Brush® Combi into the vial with preservative fluid and follow the instructions given by the manufacturer of the Pap Preparation method.



5a. Reseal the vial.



Conventional (B)

4b. Transfer the sample to the microscope slide with a painting action, applying first one side of the bristles and then the other.



5b. Immediately apply fixative to the slide. Keep a distance of approximately 30 cm (12") between the slide and the special fixative spray bottle. The fixative should be designed to deliver a very fine mist vs. a jet of fixative. The latter can result in cells blown from the slide, resulting in a less than adequate smear. Place the slide in the Rovers® Slide Mailer.



6. Complete the patient requisition form and send the vial or the Rovers® Slidemailer and the form to the laboratory

**CAUTION:**

Do not reuse the Rovers® Cervex-Brush® Combi.

The Rovers® Cervex-Brush® Combi is latex-free.

For use by health care professionals only.

Follow applicable guidelines, CDC, etc, for safe collection of human specimens.