



Push bottom of upright pole into the base.




A bag with the bolt, screws and washers are packed with the basket.


Take the bolt, thread on the split washer then the flat washer and push these through the centre of the round weight.



Turn the pole and base upside down.

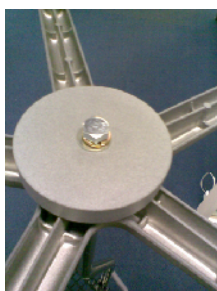


Place the round weight assembly onto the base and bolt firmly into place using a spanner.

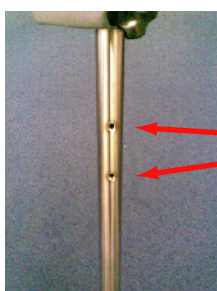

Turn the base and pole back around so the wheels are on the floor and locate the basket screw holes on the upright pole.



Attach the basket with the 2 screws using a Phillips head screwdriver

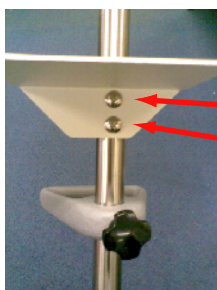

Check these screws are tight and tighten with the screwdriver if necessary