

Tono Tip Covers

Bulk Tip Covers

Powderless Softip Covers for use with the Accutome AccuPen® and Tono-Pen®.

Use before measuring intraocular pressure by covering the tip of the AccuPen® or Tono-Pen® with a Softip Cover. The Softip cover is applied by sliding onto the pen tip making certain that the rubber is flat across the tip but not taut.

It is imperative that a fresh tip cover is used for each patient to protect the eye and the instrument tip in preventing the transfer of infection. Tip covers should be replaced after examination of each patient.

This product contains natural rubber latex which may cause allergic reactions.



AO0610 Tono Tip Covers, Non-sterile, Unsleeved



AO0612 Tono Tip Covers, Non-sterile, Sanitised, Sleeved

Code	Description
AO0610	TONO TIP COVERS, NON-STERILE, UNSLEEVED (BX 600)
AO0612	TONO TIP COVERS, NON-STERILE, SANITISED, SLEEVED (BX 600 - IN BAGS OF 100)

Prepared by: Paragon Medical Pty Ltd trading as LR Instruments ABN 76 133 072 107
 Unit 7/100 New Street (PO Box 458) Ringwood VIC 3134
 Contact Details: P (03) 9870 7066 | F (03) 9870 1638 | E info@lrinstruments.com.au
 File Name: AO06_Tono-Tip-Covers_v1-0



First Prepared: 22/01/15 Eevien Tan
 Release Approval: 22/01/15 Lee Carter
 Version Control: V1.0 Page 1 | 1

Disclaimer: All images are included for reference only. Features and specifications are subject to change without prior notice.
 Goods supplied may vary from those illustrated due to continual improvements and/or regulation changes. E & OE
 Visit our website at: <http://www.lrinstruments.com.au>