



CLINIPAK SUPER SOFT SOAKER SHEETS



High rapid absorption, built-in strength

A softened version of the Clinipak soaker sheets offering high and rapid absorption with built-in strength.

Material

Absorbent Soaker Sheets are produced from pure ECF (Elemental Chlorine Free) wood pulp, manufactured with low chloride and sulphate content and free from optical brightening agents to ensure suitability as a medical paper.


The material is low linting and has excellent wet and dry mechanical properties to provide protection during the sterilisation process and subsequent handling.

Absorbency is higher than standard absorbent crepe liner and ensures condensate is quickly dispersed to provide rapid evaporation. This greatly facilitates the production of dry packs and assists the steam sterilisation process.

The sheets can be used as a heavy duty tray liner inside a tray or more commonly used under a tray to protect sterilisation wraps from internal damage while still maintaining their condensate absorbing qualities. The sheets may also be used as bath mats or for drying instruments or trays after sterilisation.




May be used as a heavy duty tray liner inside a tray.

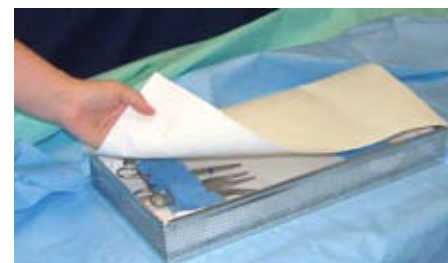

Can be used for containing spills, and also as a bath mat.



Often used between layers to facilitate dry packs.

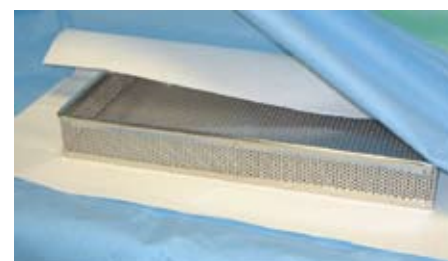

Most commonly used under trays to protect wraps from sharp corners on trays and baskets while also providing rapid absorption of condensate during processing.

Code

CSOK001SS
CSOK002SS
CSOK003SS
CSOK004SS
CSOK005SS
CSOK006SS
CSOK007SS
CSOK010SS

Description

CLINIPAK SUPER SOAKER SHEET 750 X 500 MM (PK 50)
CLINIPAK SUPER SOAKER SHEET 500 X 375 MM (PK 100)
CLINIPAK SUPER SOAKER SHEET 305 X 610 MM (PK 100)
CLINIPAK SUPER SOAKER SHEET 250 X 500 MM (PK 100)
CLINIPAK SUPER SOAKER SHEET 500 X 500 MM (PK 100)
CLINIPAK SUPER SOAKER SHEET 250 X 250 MM (PK 100)
CLINIPAK SUPER SOAKER SHEET 280 X 500 MM (PK 100)
CLINIPAK SUPER SOAKER SHEET 510 X 910 MM (PK 50)

All images are included for reference only.

Goods supplied may vary from those illustrated due to continual improvements and/or regulation changes.

E&OE First Published: 1 June 2009
Last Updated: 7 November 2011
File Name: info_CSOK_CLINIPAK_SOAKER-SHEETS

