

Honan Balloon

Intraocular Pressure Reducer

The best way to lower intraocular pressure before surgery!

Pressure on the eye traditionally has been a method of slowing some tachycardias. Some persons may be more sensitive to the ocular-vagal reflex. Therefore, all patients should be monitored for signs of bradycardia while pressure is being applied to the eye.

The optimum pressure to be used should be well below pressure in the central retinal artery. It should be elevated only high enough to create a soft surgical eye. Using the 20 to 30 mm Hg of monitored pressure for 30 to 60 minutes before surgery, clinically results in very soft, safe, surgical eyes. With the Schiotz Tonometer with 5.5 gram weight, the scale reading may be above 10. An eye may be so soft that the cornea may be observed to dimple when the lid speculum is inserted. Releasing the ressure every 30 seconds and then reapplying it has NOT been found necessary or desirable.

With a soft eye, the iris may be concave or drop posteriorly after lens removal. Excess vitreous pressure is typically absent. Intraocular lens implantation is much easier and safer. From the surgeon's viewpoint, there is much less stress and strain.



Code	Description
HB2050	HONAN INTRAOCULAR PRESSURE REDUCER REUSABLE
HB2055	HONAN INTRAOCULAR PRESSURE REDUCER REUSABLE WITH RELIEF VALVE
HB3000	HONAN INTRAOCULAR PRESSURE REDUCER DISPOSABLE (BX 12)
HB3005	HONAN INTRAOCULAR PRESSURE REDUCER DISPOSABLE WITH RELIEF VALVE (BX 12)
HB150	HONAN BELLOWS WITH SIDE INLET
HB200	HONAN SINGLE HEADBAND REUSABLE
HB210	HONAN DIVIDED HEADBAND STANDARD REUSABLE
HB300	HONAN TUBING BLACK LATEX
HB325	HONAN SILICONE TUBING LATEX FREE
HB400	HONAN RELIEF VALVE
HB500	HONAN PRESSURE GAUGE WITH BLACK BULB BLACK LATEX
HB505	HONAN PRESSURE GAUGE WITH BLUE BULB LATEX FREE
HB550	HONAN BLACK BULB AIR PUMP BLACK LATEX
HB555	HONAN BLUE BULB AIR PUMP LATEX FREE
HB600	HONAN CASE BLACK VINYL WITH ZIPPER
HB650	HONAN ADAPTER REUSABLE
HB701	HONAN SINGLE USE DISPOSABLE BELLOWS & HEADBANDS (BX 12)
HB703	HONAN SINGLE USE DISPOSABLE BELLOWS & HEADBANDS (36 - 3X BX 12)
HB706	HONAN SINGLE USE DISPOSABLE BELLOWS & HEADBANDS (72 - 6X BX 12)



Prepared by: Paragon Medical Pty Ltd trading as LR Instruments ABN 76 133 072 107
 Unit 7/100 New Street (PO Box 458) Ringwood VIC 3134
 Contact Details: P (03) 9870 7066 | F (03) 9870 1638 | E info@lrinstruments.com.au
 File Name: HB_Honan-Balloon_v1-0

First Prepared: 22/01/15 Eevien Tan
 Release Approval: 22/01/15 Lee Carter
 Version Control: V1.0 Page 1 | 1

